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EN Follow instructions for use

nexsana® SITZKISSEN SEAT CUSHION **EN**

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****EN** nexsana® seat cushion. Ergonomic and supportive seat cushion made of certified memory foam with soft, breathable and washable fabric cover in anthracite.**

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****EN**** Thank you for choosing the nexsana® seat cushion and for the trust you have placed in it. The nexsana seat cushion is a high quality U-shaped seat cushion made of certified memory foam with a soft and at the same time breathable, washable fabric cover in anthracite to maintain, support and promote a correct sitting position, relieve pressure on the buttocks, lower abdomen and back and can help to reduce pressure in the anal region. Further information and application tips can be found on the back.

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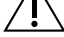
EN Cover gently washable at 30 °C.
Do not use a dryer. Do not iron.

EN The nexsana® seat cushion is an ergonomic and supportive seat cushion. The hardness of the CertiPUR-US-certified high density visco elastic memory foam is designed in such a way that an optimally balanced, supportive and at the same time comfortable distribution of the body weight is made possible, thus avoiding both sitting through and pressure pains. Due to the special composition, the extended weight range is between approx. 40 and 110 kg. The mesh fabric selected for this seat cushion is breathable and at the same time very soft and gentle on the skin, can be detached from the seat cushion with a zip and gently machine washed at 30 °C. Do not wash the seat cushion as a whole.

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EN Secure seat cushion from children and keep away from heat and fire. 

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EN If you still have questions, please feel free to contact nexsana customer service.

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nexsana GmbH